AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and closely observe the specifications in these instructions for use and the instructions given by the specialist. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

AchilloTrain is a medical device. It is a support for the relief of Achilles tendon complaints with a friction pad and a heel wedge.

 **AchilloTrain must be fitted for the first use by a trained specialist*, otherwise we cannot accept any liability for the product.**

Indications

- Achillodynia (tendinosis, paratendinitis, bursitis subachillea)
- Haglund's deformity
- Chronic, post-traumatic or post-operative irritation (e.g. in the case of Achilles tendon ruptures)

Risks of using this product

Important Information

The benefits of this product are felt particularly during physical activity.

- Please remove the support during longer periods of rest.
- Once you have been prescribed the AchilloTrain, only use it according to your indication and any additional instructions given by a medical specialist. Please consult a specialist or your doctor first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your doctor.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. In particular, the ability to operate vehicles, other means of transport or machinery can be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Furthermore, proceed with extreme caution, as your ability to stand and walk is also heavily impacted.
- Check the specific ground characteristics and ensure the ground is not slippery whenever walking, standing or otherwise stressing the affected part of the body. If the ground is smooth, there is an acute risk of slipping and thus injury!

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date.

In the following conditions such aids should only be fitted and worn after consultation with your doctor:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the foot, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting AchilloTrain

1. A donning aid is included to allow for easier donning of the support. For donning, please follow the instructions provided in the quick reference guide.
2. Insert your thumbs into the palpable grip recesses on the inside of the upper edge of the support and pull it over the foot until the heel is positioned in the blue knitted section **1 + 2**.
3. Ensure that the Achilles tendon is securely held by the shaped insert.
4. Adjust the fit of the support as necessary.
5. If you require stronger relief of the Achilles tendon, use the included heel wedges in your shoes. Always use both heel wedges on both sides (not just on one foot) to avoid misalignments due to one-sided elevation. Place the heel wedges in your shoes so they fit snugly into the heel area of the shoes **3**.
6. To take off the support, take hold of the grip recesses on the upper edge again and pull the product over your foot from the upside

down so that the support is reversed after taking it off.

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product. If required, clean the heel wedges with a damp cloth. Do not wash the heel wedges in the washing machine.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the AchilloTrain Pro have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.). If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI), Thermoplastic elastomer (TPE), Polyamid (PA), Spandex (EL), Cotton (CO), Polyurethane (PUR), Polyester (PES)

 – Medical Device
 – DataMatrix Unique Device Identifier UDI

Version: 2023-10

* A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

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